



INSTRUCTIONS FOR USE

1. Wash and disinfect your hands thoroughly before use. Make sure the product is intact and that the product has not expired.
2. Put the surgical mask on your face by the straps, not touching the surface of the mask.
- 3.-4. Carefully shape the nose strip in the upper edge of the mask over your nasal bridge and pull the lower edge of the mask under your chin.
5. Use the mask continuously. Do not remove it temporarily during use. Replace the mask with a new one if you need to remove it or it gets damaged.
6. Do not touch the outer surface of the mask. The mask needs to cover mouth and nose throughout use. Never pull it down, for example to hang around your neck.
7. Wash and disinfect your hands before removing the surgical mask. Remove the mask by the straps.
8. The surgical mask is disposable. Dispose of the mask in accordance with work place practice immediately after use. Wash and disinfect your hands again after this.