




NeSu[®]
PROFESSIONAL
UltraSoft -face mask

SINGLE-USE SURGICAL MASK
EN 14683:2019 + AC:2019 | Type II
Model UL90-TII | Adult size

Manufacturer:
Filterpak Oy Ab Ltd
Hangonkyläntie 67 10900 Hanko Finland
www.filterpak.fi | www.nesu.fi

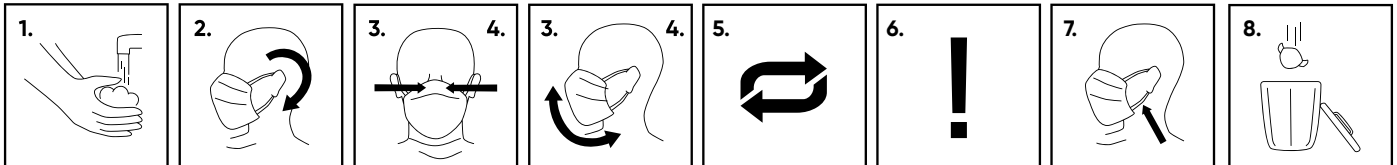

Päiväys / Datum / Date: 22.10.2020

INSTRUCTIONS FOR USE

1. Wash and disinfect your hands thoroughly before use. Make sure the product is intact and that the product has not expired.
2. Put the surgical mask on your face by the straps, not touching the surface of the mask.
- 3.-4. Carefully shape the nose strip in the upper edge of the mask over your nasal bridge and pull the lower edge of the mask under your chin.
5. Use the mask continuously. Do not remove it temporarily during use. Replace the mask with a new one if you need to remove it or it gets damaged.
6. Do not touch the outer surface of the mask. The mask needs to cover mouth and nose throughout use.
7. Wash and disinfect your hands before removing the surgical mask. Remove the mask by the straps.
8. The surgical mask is disposable. Dispose of the mask in accordance with work place practice immediately after use. Wash and disinfect your hands again after this.

NOTE! Proper use of the product at all stages of use is important. If the instructions are not followed, the user of the product may be exposed to health hazards that occur during use. If you use other protectors with the surgical mask, make sure that the products used at the same time do not interfere with each other's safe operation. Do not use the product if it has expired. Do not touch the surgical mask during use. The surgical mask is for single use only. Do not wash or reuse the product! The product or its material is not known to cause irritation or inconvenience to the user. Consult a doctor before using surgical masks in case of lung disease or other breathing difficulties. The product is not a personal protective equipment or respirator. The product has not been tested for use in potentially explosive atmospheres. The flammability of the product has not been tested. It must not be used in the vicinity of open flames.

Store the surgical masks hygienically in a dry place at a temperature between +5 - +45 C and relative humidity less than 80 %.

Material index: Polypropylene, acrylic, elastane, nylon, polyethylene, iron wire

